

Hybrid Bond ONE

Dental bonding agent

1 Bottle

1 Step

Bonding agent




STEPS




Apply 20 sec

Please keep the surface wet for 20 sec for better bond strength.

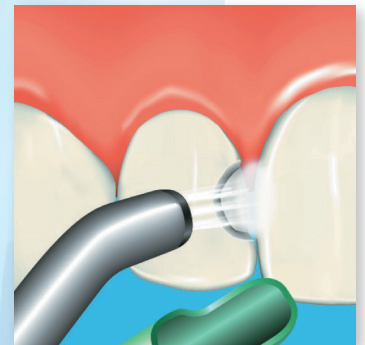

Air Blow 5~10 sec

Air blow the surface, using vacuum to prevent Hybrid Bond ONE from scattering.


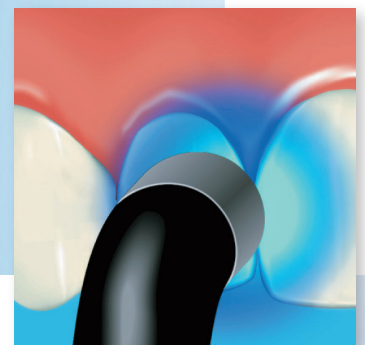


Light-cure 5 sec

Irradiate light for polymerization.

Remove water effectively


Water is effectively transported by acetone's evaporation.

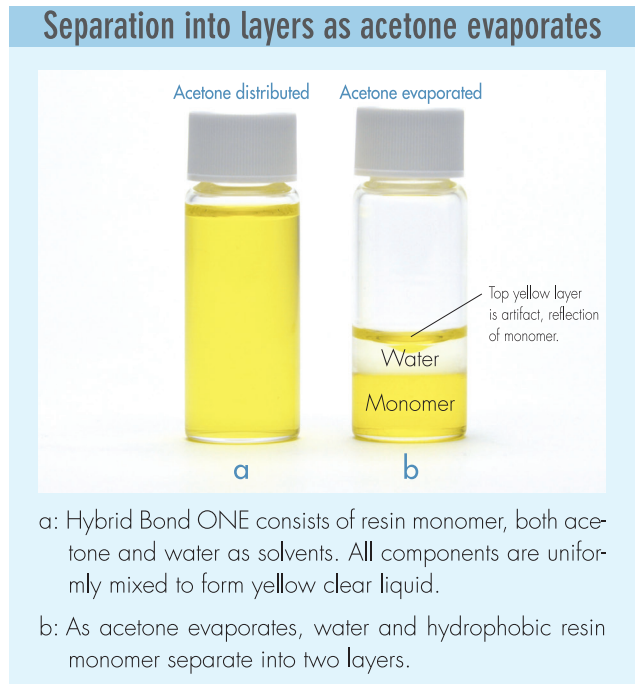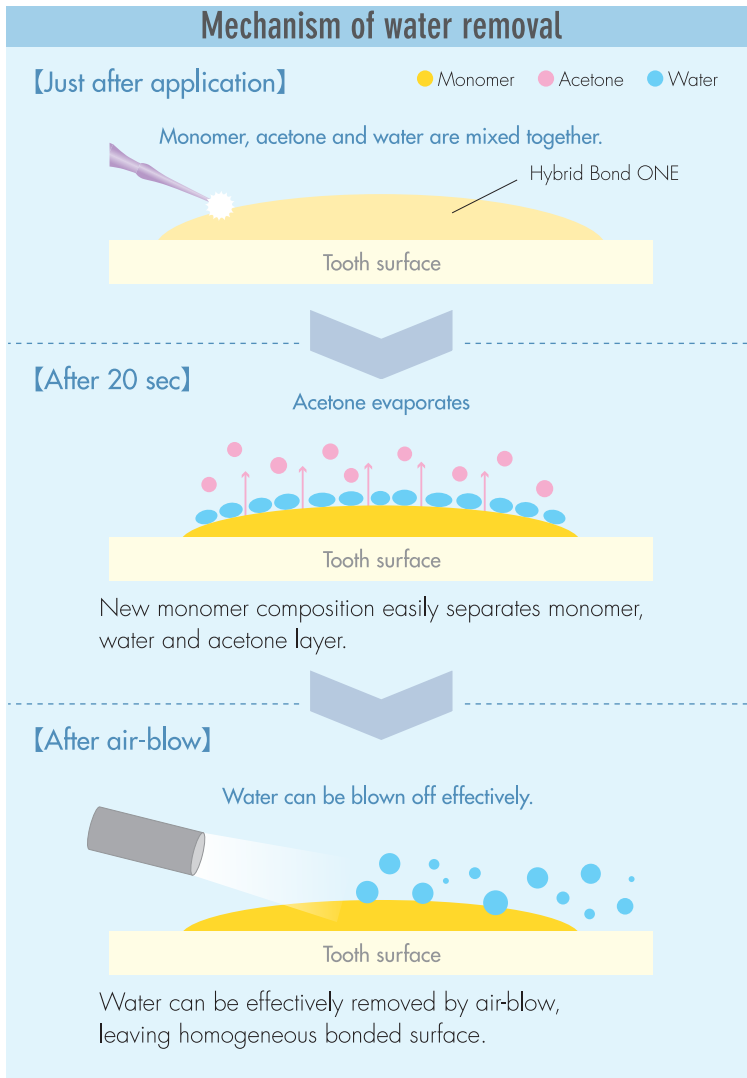


Durable bonding performance

Hybrid Bond ONE has same bonding performance with both enamel and dentin.

【Shear bond strength to bovine enamel】

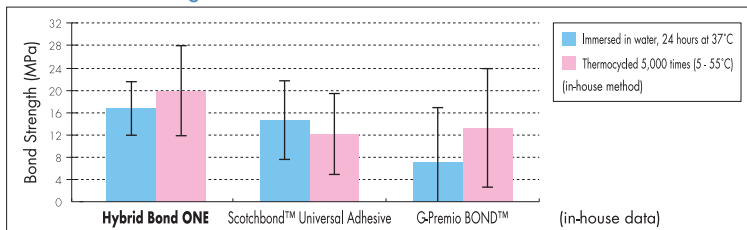


【Shear bond strength to bovine dentin】

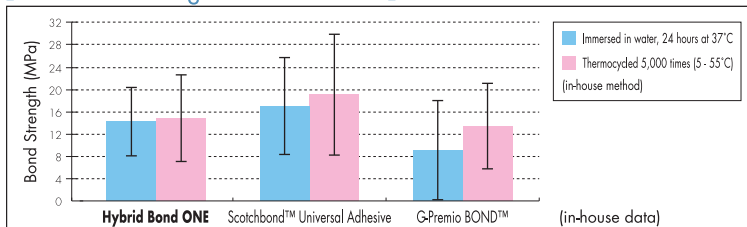

Thin adhesive layer: Only 5 μm

Suitable for cases require high aesthetics.

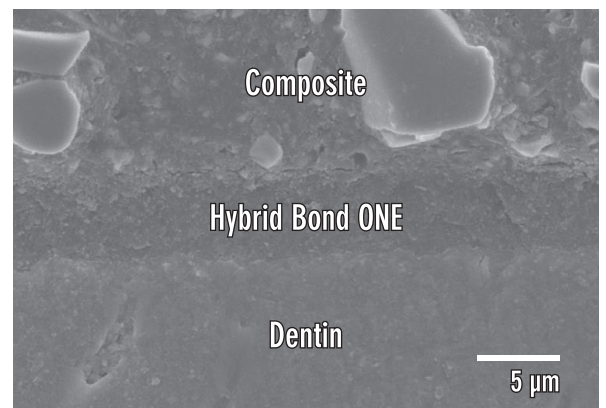

Only 5 μm of thin layer has excellent aesthetics.